Material
Polyamid (PA)
Glasfaser (GL)
Engage Compound (EVA)
Polyurethan (PUR)

Material
Polyamide (PA)
Fiberglass (GL)
Engage Compound (EVA)
Polyurethane (PUR)

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Stand der Information: Oktober 2010

EN Dear customer,

thank you for choosing a product from Bauerfeind. ValguLoc® II is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your physician, medical retailer or our Technical Service department.

A crooked big toe, medically known as hallux valgus, is a condition that commonly accompanies splayfoot. The condition can be exacerbated for example by high heels or too tight-fitting shoes and can result in painful inflammation of the bursa on the ball of the foot. ValguLoc® II is designed to bring the big toe into a therapeutically favorable position. To this end, ValguLoc® II can be adjusted to a variable correction angle. ValguLoc® II also incorporates a joint that permits physiological movement of the metatarsophalangeal joint and thus allows the wearer to walk normally in comfortable, wide-fitting shoes.

When used immediately after a hallux valgus operation, your doctor can lock the movable joint in order to immobilize the toe for a short period. In this case the patient may only walk in special footwear (e.g. orthopaedic shoe with a stiffened sole).

ValguLoc® II must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis¹.

Indications

- Misalignment of the big toe (Hallux Valgus)
- Postoperative protection or conservative treatment

Side effects

Side effects involving the body as a whole have not been reported to date. Correct use/fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs and postoperatively to impaired wound healing or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Wounds at sites where the splint comes into contact with the skin.
2. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling/redness and excessive heat build-up.
3. Impaired sensation and circulatory disorders of the feet, e.g. Diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.
5. Gout in the big toe (podagra).

Fitting ValguLoc® II

The dynamic big toe splint can be worn on the right or left foot. When fitting the splint, ensure that the marking for the respective side appears at the top of the scale. If fitted to the right foot, the R (for right) in light-grey lettering should appear at the top. If fitted to the left foot, the L (for left) in red lettering should appear at the top. If right and left are mixed up the splint may slip, potentially resulting in adverse effects on the metatarsophalangeal joint and the loss of the therapeutic effect.

As supplied from the factory, the joint stop of ValguLoc® II is adjusted for use after surgery. To release the stop proceed as follows:

1. Fold back the inner pad over the joint to reveal the inner locking ring with two lugs on opposite sides (Fig. 1).
2. Remove this locking ring, which is not required for conservative treatment. Walking with the splint is not possible when the locking ring is inserted as the joint remains immobile (Fig. 2). You will find it easier to remove the inner locking ring if you remove the outer locking ring beforehand (Step 5, Fig. 5) and then push the inner locking ring inwards from the outside.
3. Now position the splint on the inner aspect of the foot to be fitted. Open the upward-pointing straps and pull these out of the guide slit. Ensure that the toe shell is located on the side of the foot. The splint joint on the inside of the foot must be centered precisely over the middle of the metatarsophalangeal joint (ball) (Fig. 3).

¹ Orthosis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso

4. Place the small strap around the big toe starting from the bottom. Position the large strap beneath the foot. Close both straps by pulling the toe strap around the big toe starting from the bottom, passing it through the guide slit and fasten lightly. Pass the foot strap around the midfoot, again starting from the bottom, pull it through the guide slit and fasten lightly. Now undo the end of the foot strap and tighten it until the splint fits firmly against the foot but without any uncomfortable pressure. Now do the same with the toe strap (Fig. 4). As supplied, the toe strap is long enough to fit around a bandaged toe after a hallux valgus operation. If used without a bandage the strap can be shortened during this adjustment (trim Velcro® fastening with scissors).

The splint is correctly fitted to the foot if the metatarsophalangeal still retains most of its mobility. If so, it may now be worn inside a shoe. To avoid unnecessary pressure points, you should only wear the splint in a wide-fitting comfortable shoe (e.g. sport shoe).

If the splint is used for immobilization after an operation, any movement in the metatarsophalangeal joint must be prevented (lock the splint joint). If used postoperatively, the splint may only be applied in an orthopaedic shoe with a stiffened sole.

Adjusting and locking the correction angle

An eccentric adjuster on the support can be used for the external graduated adjustment of the varus-valgus angle (in approx. 2° increments from 10° valgus to 2° varus; setting as supplied from the factory: +6°). For a splint worn on the right foot, the angle must be adjusted in the R direction (grey scale), while a splint worn on the left foot must be adjusted in the L direction (red scale). Proceed as follows:

5. Remove the outer locking ring of the splint using the supplied adjuster chip (Fig. 5). Press the flattened side of the adjuster chip against the locking ring from the bottom in an upward direction to release the ring from its anchorage and reveal the adjuster ring.
6. Remove the inner locking for the joint stop as described in steps 1 and 2 and fit the splint to the respective foot as described in steps 3 and 4. The correction angle can now be adjusted on the fitted splint (Fig. 6). If the foot is painful or postoperative, the correction angle should not be adjusted while the splint is fitted to the foot.

7. To adjust the angle, engage the supplied adjuster chip (or a suitable coin) in two slots on opposite sides and turn the ring until the pointer shows the desired angle (Fig. 7).
8. To lock the adjusted angle, press the outer locking ring back into the adjuster ring (Fig. 8) by locating the notch in the outer locking ring over the pointer in the adjuster ring and pressing the locking ring firmly until it clicks into place (Fig. 9).

⚠ Instructions for the orthopaedic technician

Joint locking after surgery or other indication

- To lock any movement in the splint joint, the locking ring must be clicked in place. As supplied, the splint is locked in 0° flexion.
- Release the joint stop as described under steps 1. and 2.
- Before locking the joint in the desired position, undo the straps and hold the splint in your hand. Locate the splint laterally next to the respective metatarsophalangeal joint and note this position.
- Now fold back the anterior pad on the medial side. You will now see a toothed grating into which the internal locking ring can click into place (Fig. 2). Adjust the desired angle of flexion and engage the locking ring in the toothed grating. Ensure that the two lugs of the locking ring fit into the locking notches. The hallux valgus joint is now locked.
- Fold the pad back to its former position and refit the splint on the patient.
- The splint can now be used in its locked setting.
- To release the lock after a few days, remove the internal locking ring as described in steps 1 and 2 and, at the same time, check the correctness of the varus/valgus angle.
- Finally, pull off the white release paper on the inside of the anterior pad and press the pad against the toe shell. The pad is now stuck to the toe shell. This prevents slippage of the toe pad during movement.

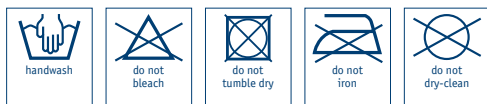
Splayfoot pad

If necessary, the splint can be used in combination with a splayfoot pad. In this case, select an adhesive pad that ensures secure positioning inside the shoe.

⚠ Important information

- ValguLoc® II is a prescribed product that should be applied under a physician's guidance. ValguLoc® II should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- Do not walk while wearing the product.
- It should only be used when the foot is not placed under load.
- No liability is accepted for improper use of the product.
- Improper changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care and washing instructions

The support can be washed separately by hand using a mild alkali-free detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

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