medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
A-6020 Innsbruck
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Voortstraat 79
3580 Beringen / Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Danmark
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona/Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL/England
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Polska Sp. z o. o.
Chełmońskiego 1a
44-100 Gliwice
Poland
T+48 32 230 60 21
F+48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex /France
P +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi othec b.v.
Heusing 5
NL-4817 ZB Breda/Netherlands
P +32 11 24 25 60
F +32 11 24 25 64
mediortho@medibelgium.be
www.medibelgium.be

medi Bayreuth Unipessoal, Lda
Rua Eugénio de Castro Rodrigues, nº2
1700-184 Lisboa/Portugal
P +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt

medi RUS LRC
Business Park "Rumyantsevo"
Rumyantsevo, Build. 1
Leninsky Township
142784 Moscow Region/Russia
T +7 495 229 04 58
F +7 495 229 04 58
info@medirus.ru
www.medirus.ru

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000/USA
P +1 336 4 49 44 40
F +1 888 5 70 45 54
orders@mediusa.com
www.mediusa.com

medi Middle East
P. O. Box: 109307
Abu Dhabi
United Arab Emirates
P +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi

ULTIMATE
COMPRESSION
TECHNOLOGY

Lumbamed® plus

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Lumbamed® plus

Dear Patient

We are delighted that you have decided to use a top-quality product from medi and thank you for the trust you have shown in us.

Please read these instructions carefully before using the support for the first time. If you apply the support correctly it will develop its complete medical efficacy and will be comfortable to wear – so that you will feel better soon.

Areas of use

- Non-specific pain in the area of the lumbar spine (LS)
- (Repeated) Pain due to attrition in the area of the LS, e.g. on account of changes to the intervertebral disks
- Irritations of the sacroiliac joint
- Overstrain injuries in the transitional area lumbar spine/pelvis
- Weak or unbalanced trunk muscles (back, abdomen)

Side-Effects / Contraindications

No side-effect is known which influences the whole body. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances before applying the support:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of action / Comment on wear

Lumbamed plus has a stabilising and relieving effect on the lumbar spine. Its compressive knitted fabric and the pad that is used also stimulate circulation and have a pain-relieving, massage effect without affecting the muscular force.

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin (Comfort Zone) is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialized shop.

The support is suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / Taking off

- Pull the textile cover over the massage pad and fix in the middle on the central, grey Velcro area between the two stays according to the instructions of your doctor or a specialist. (Abb. 1) The pad should be fastened so that its lower end rests just above the fold between the buttocks when worn.
- First slip your left hand into the internal wrist strap of the left side of the fastener. Guide the bandage round the body and then slip your right hand into the external hand strap of the right side of the fastener (Fig. 2 + 3)

- Pull both sides of the fastener into the middle so that they overlap as far as possible and stick to each other by the Velcro pads as evenly as possible. The bandage is properly fitted when the lower edge of the fastener is just above the groin (Fig. 4 + 5)

Please always wear the Lumbamed plus on the skin and under your clothing. This maximises the Clima Comfort Effect. Please take care that trained staff instruct you when putting on the support for the first time.

Instruction for washing / Care tips

Velcro tabs must be closed for washing. Do not wash the pad with the support. It can be cleaned with a damp cloth. Wash the product separately with a woollen or mild detergent up to 30° in cradle wash action or by hand. Rinse out the product properly and hang it up damp. Do not dry it in the dryer, on the radiator or directly in the sun. Do not remove spots with benzene or chemical detergents. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

Note on Storage

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Composition of the Material

Support: Polyamide, elastan

Pad: Polyurethane, polyethylene

Statutory Defects Liability / Contractual Guarantee

Consistent with legal guidelines.

Important Notes

It is recommended that the support should only be used for single patient. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer will be forfeited according to the Medical Product Law.

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